Add

10 ml

CONSOL

04113
DPK-00023-11
2021-05-10

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EN Instruction

Add1 is a preservative-free viscous comfort solution that lubricates and moistens the eye. Add1 has been developed to induce natural tear film recovery, thereby preventing dry eye symptoms such as burning, runny, tired eyes and foreign body sensation. Add1 can be used with contact lenses of any type of material.

Instructions for Add1 pump bottle

1. Remove the protective cap.
2. Before using Add1 for the first time, invert the bottle to a vertical position with the tip pointing downwards. Then, repeatedly press the bottom of the bottle until the first drop leaves the tip. The bottle is now ready for use.
3. To dispense one drop of Add1 first place the bottle between your thumb and index finger, with the tip of your thumb pressing against the ribbed surface around the tip, and the index finger on the bottom of the bottle. To reinforce your grip place the middle finger next to your index finger.
4. Without tilting your head backwards and with the bottom of the bottle pointing upwards, pull down the lower eyelid. Apply one drop into the inside of the lower eyelid, on the outer corner of the eye, i.e. away from the nose. Blink repeatedly to evenly spread the eye drop over the surface of the eye.
5. After use, make sure the tip is dry by flicking the bottle and then cover by putting the protective cap back on. To avoid contamination, never let the dropper tip come in contact with any surface.

Important when using Add1

- Do not use Add1 if the bottle is broken or open in any way.
- The grip of the bottle may worsen if the solution comes into contact with the fingertips. Wash and dry the hands before further use.
- Do not use as a storage solution for contact lenses.
- Do not tilt your head back when applying Add1.
- Avoid dropper tip contact with any surface.
- Store Add1 in a refrigerator or at room temperature.
- Add1 pump bottle: If the product is left unused for a long period of time or if you try to use Add1 without the bottle being inverted to a vertical position with the tip pointing downwards, the pump needs to be primed again. To prime it, follow the above instructions for using Add1 for the first time.
- Do not use after expiration date.
- Add1 can be used for up to 6 months after first use.
- Keep out of reach of children.
- When using Add1 combined with other solutions, gels or ointments that are intended for ocular use, e.g. eye drops, eye gels, eye ointments, contact lens solutions, or newly inserted contact lenses - wait a minimum of 30 minutes before applying Add1 to your eyes.
- Do not use if you are allergic to any of the ingredients in Add1.
- Contact your ophthalmologist or optician if you are requiring more than ten drops a day.
- Possible risks, side effects or complications connected to the use of eye drops are: allergic reaction, inflammatory reaction or infection. If you experience any problems contact your optician or ophthalmologist.

Content

A sterile isotonic solution that contains purified water, sodium chloride, potassium chloride, calcium chloride, magnesium chloride, citrate buffer, sorbitol and sodium hyaluronate 2,1 mg/ml. Does not contain BPA (Bisphenol A), phthalates, nor borate buffer.

Instructions for Add1 squeeze bottle

1. Tear off the tamper-evident seal.
2. With your index finger around the bottle neck, your thumb pressed against the ribbed surface and the hand gripping the bottle, press your thumb upwards and against the lid, until it comes loose.
3. To dispense one drop of Add1 Squeeze, squeeze the bottle with the bottom pointing

- upwards.
4. Without tilting your head backwards and with the bottle pointing downwards, pull down the lower eyelid. Apply one drop into the inside of the lower eyelid, on the outer corner of the eye, i.e. away from the nose. Blink repeatedly to evenly spread the eye drop over the surface of the eye.
5. After use, make sure the tip is dry by flicking the bottle and then cover the tip by putting the protective cap back on. To avoid contamination, never let the dropper tip come in contact with any surface.

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