
EXTEND 100 ml, 355 ml

instruction / bruksanvisning / brukerveiledning / käyttöohjeet

CE 0413
2021-08-18
DPK-0000044-15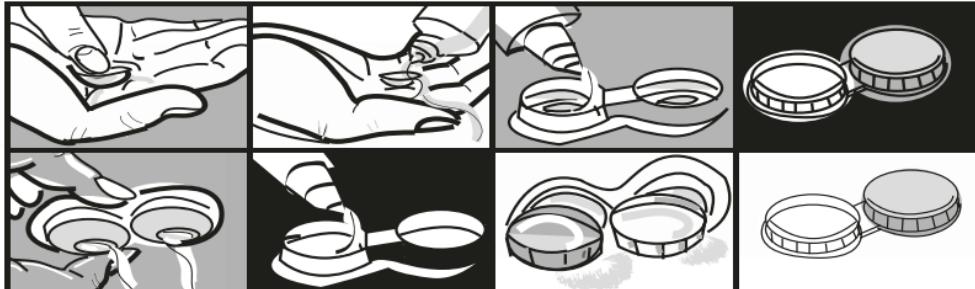

EN

EXTEND is a solution used for cleaning, rinsing, disinfection and storage of contact lenses of any type of material. With each 355 ml bottle follows one lens case.

Instruction

1. Wash your hands.
2. Remove one lens at a time and place it in the palm of your hand.
3. Clean the lens by adding EXTEND in abundance, then rub it.
4. Rinse with EXTEND.
5. Place the lenses in the lens case. Fill with a minimum of 2/3 of EXTEND.
6. Tighten the cap and disinfect for at least 4 hours.
7. Before using the lenses, gently pour the solution from the case, leaving the lenses on the edge of the wells to be easily taken out.
8. Rinse the lens case with EXTEND and let air-dry when unused.
9. Lenses can be stored in a fully closed lens case for 30 days maximum. Then disinfect again according to step 1-6 above.

Important

- For best possible hygiene; replace the old lens case with a new one, with each new bottle.
- In order to avoid any serious complications; always follow the instruction of your contact lens practitioner.
- If irritation occurs; discontinue lens wearing and contact your contact lens practitioner.
- Never reuse EXTEND solution.
- Always keep the solution out of reach of children.
- The solution is sterile before the bottle is opened.
- The maximum period of use after the bottle has first been opened is 6 months.
- Always make sure to close the bottle carefully directly after using it.
- Do not use EXTEND after expiry date.
- Do not use if the bottle is damaged.
- Store at room temperature.

Contents

A sterile isotonic solution containing borate buffer, disodium EDTA and sodium chloride; active ingredients: chlorhexidine gluconate 0.001%, poloxamine 1%.

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